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Referencias 1. Smith & Nephew 2016. ALLEVYN Gentle Border Heel Gen 2 - physical evaluation. Internal Report. DS/16/465/R. 2. Rafter L, Reynolds T, Rafter M. An audit of patient outcomes in the management of skin tears using silicone dressings. Wounds UK. 2016;12(2):70-78. 3. Hampton S. An evaluation of a silicone adhesive shaped heel dressing. Br J Nurs. 2010;19(6):S30-33. 4. Hurd T, Gregory L, Jones A, Brown S. A multi-centre in-market evaluation of ALLEVYN Gentle Border. Wounds UK. 2009;5(3):32-44. 5. Smith & Nephew 2017. ALLEVYN Gentle Border Gen 2 - Physical Evaluation. Internal Report. DS/16/424/RV3. 6. Callaghan R, Chadwick P, Mahoney K, Stephen-Haynes J. Case series evaluation: The use of Allevyn gentle border multisite on chronic wounds. Wounds UK. 2012;8(4):110-118. 7. Smith & Nephew 2018. Volunteer Evaluation of Wound Care Dressing Product. Features. Internal Report. AWM/AWC/001/v1. 8. Smith & Nephew 2017. An open, prospective, randomised, comparative study to compare the performance of ALLEVYN Gentle Border Multisite with an alternative silicone adhesive dressing. Internal Report. GMCA-DOF-09. 9. RD/17/103 - Technical comparison of ALLEVYN Gentle Border and ALLEVYN Life to assess the impact of a wound contact layer change. 10. Smith & Nephew Data on File GMCA-DOF-09 An open, prospective, randomised, comparative study to compare the performance of ALLEVYN Gentle Border Multisite with an alternative silicone adhesive dressing. 11. Smith & Nephew Wound Management. Data on File Report Reference OR-DOF/017, An open, prospective, randomised, comparative study to compare the performance of ALLEVYN Gentle Border Multisite with an alternative silicone adhesive dressing. Palmer A and Mistry C; June 2011. 12. Coggins T, Doubleman J, Gray C, Ray W, Slowikowski G. Using a hydrocellular foam dressing with silicone adhesive* as part of a comprehensive pressure ulcer prevention plan: Results from five US hospital ICUs. Paper presented at: SAWC, 2012. 13. Moody A, Bielby A. An evaluation of a silicone-based adhesive foam heel dressing (ALLEVYN Gentle Border Heel). Paper presented at: Wounds UK 2009; Harrogate, UK. Smith & Nephew S.A.U., Fructuós Gelabert, 2-4, 08970 Sant Joan Despí, Barcelona T. +34 902 30 31 60, F. +34 902 30 31 70 www.smith-nephew.com/allevyn *Marca registrada de Smith+Nephew. Todas las marcas registradas y reconocidas. ©Diciembre 2020 Smith+Nephew AWM-AWC 25198-es V1 ALLEVYN GENTLE BORDER Heel application sheet 1220