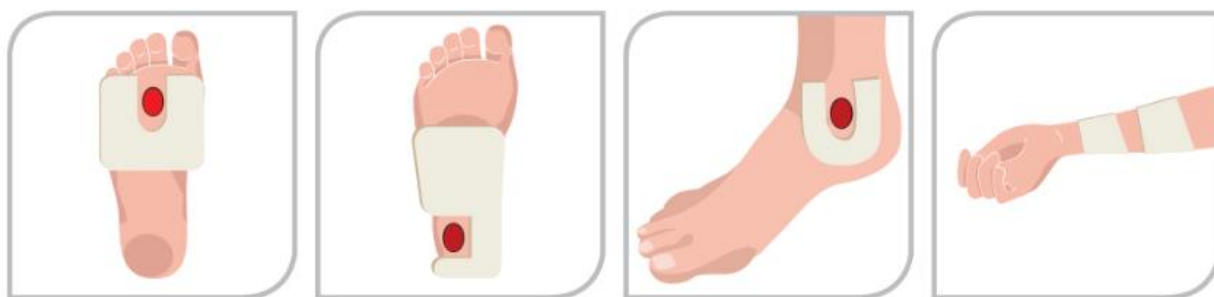
## FICHA TÉCNICA/*TECHNICAL DATA SHEET*

Fieltro 100% lana/*Felt 100% wool*  
F-00527

### DATOS DEL FABRICANTE *MANUFACTURER'S DETAILS*

Fresco International 2005, S.A.  
Calle Torre de Cellers, 5, 08150, Parets del Vallès, Barcelona, España.  
(+34) 932314812  
[info@fresco.es](mailto:info@fresco.es)  
[www.fresco.es](http://www.fresco.es)

## INDICE/INDEX

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### **DEVICE DESCRIPTION**

*Felt support surfaces to facilitate the healing of foot ulcer.*

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### **INTENDED PURPOSE**

Relieve pressure and avoid contact of the wound with any surface.

It can be used on any part of the body considered at risk, especially on body prominences.

Product is not washable or reusable.

### 2.1 End of life disposal

Replaced when:

- « Visible worn.
- « It is detached from the skin.
- « Fails to provide complete coverage of the area at risk of pressure damage due to loss of integrity.

At the end of its service life, dispose of it with non-hazardous waste.

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## **INDICATIONS**

- *It is indicated for pressure relief of neuropathic and neuroischaemic ulcers.*
- *Use in direct contact with the patient's skin, on areas of the body surrounding the area where pressure is to be avoided.*
- *A sufficiently thick felt should be used in the plantar area to redistribute pressure from the wound to other healthy parts of the foot. The thickness of the felt will depend on the weight of the patient, the heavier it is, the thicker it needs to be.*
- *The offloading is complementary to the usual wound treatment (topical and other).*
- *Fresh Felts can be used on the inside of total contact casts.*

### **Mode of action**

*A correct application of the felt redistributes the pressure from the ulcer towards the healthy plantar area.*

*The Ulcer is left in flotation (without pressure) which allows the wound to progress towards healing.*

*The felt supports the healthy parts of the foot, its semi rigid behaviour allows it to "give way" when walking or extra pressure is put on top without bottoming out, impeding the wound from contacting the floor and thus remaining pressure free.*

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### **PATIENT TARGET GROUP**

*Any person who has an ulcer or is at risk of developing one.*

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### **INTENDED USERS**

*Intended for use in home and professional healthcare facilities by medically trained and untrained users.*

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### **CONTRAINDICATIONS**

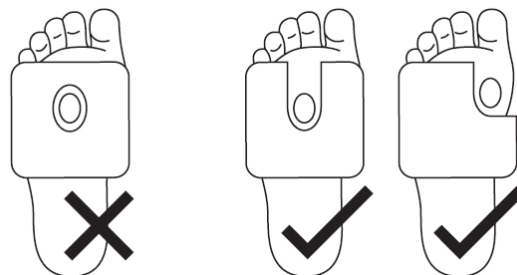
*The use of felt shocks in ischaemic ulcers is not effective as they are not caused by pressure. The practitioner must discern the correct aetiology of the ulcer.*

*ulcer, if friction and pressure are not a factor felt is of little use.*

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### WARNINGS AND PRECAUTIONS

- *Making a relief hole in the felt without a side relief opening may cause window oedema (see picture below).*
- *Ensure that the felt is thick enough to relieve pressure effectively.*
- *When the patient's weight is on the discharge area, the ulcer should not come into contact with the floor.*
- *Do not cover the ulcer with felt. Covering the fingers should also be avoided if there are no clear biomechanical reasons for doing so.*

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##### **Raw Material**

##### Felt

*Composition of 100 % wool, very smooth to the touch.*

*Colour: White*

##### Adhesive

*Acrylic with silicone paper as protector.*

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##### **Biocompatibility tests**

*Testing has been conducted in accordance with "ISO 10993-1 Biological Evaluation of Medical Devices – Part 1: Evaluation and Testing within a risk management process".*

*The specific biocompatibility test conducted are the following:*

1. *Cytotoxicity by direct contact*
2. *Skin irritation*

### 3. Maximization test for delayed hypersensitivity

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### Physical and chemical characteristics

|                         |                                              |
|-------------------------|----------------------------------------------|
| Color/Colour:           | Blanco/White                                 |
| Aspecto/Aspect:         | Sólido/Solid                                 |
| Solubilidad/Solubility: | No es soluble en agua/No solubility in water |

## 9. VARIANTES DEL PRODUCTO, DIMENSIONES Y PACKAGING. DEVICE VARIANTS, DIMENSIONS AND PACKAGING.

### Referencias References

F-00527-04C  
F-00527-06C  
F-00527-08C  
F-00527-10C  
F-00527-10CV

### Dimensiones Dimensions

Rollo/Rol 10cmx3m Thickness 4 mm  
Rollo/Rol 10cmx3m Thickness 6 mm  
Rollo/Rol 10cmx2m Thickness 8 mm  
Rollo/Rol 10cmx2m Thickness 10 mm  
Rollo/Rol 10cmx2m Thickness 10 mm

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*Label symbol, lot and date of manufacture.*

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### INSTRUCTIONS FOR USE

1. Clean the wound or affected area.
2. Remove the felt from its packaging.
3. Cut the felt into the appropriate shape to relieve the ulcer.

4. *Remove the protective paper from the back to uncover the adhesive and stick to the skin.*
5. *Once the felt has been properly applied, cover the ulcer with gauze to protect the wound and wrap the foot.*

*The felt should be changed 3-4 days after application depending on the severity of the ulcer. The more severe the ulcer, the more often it needs to be changed, also so that the medical professional performing the application has the possibility to inspect the wound and see how it is progressing.*

*The treatment should last until the wound has closed.*

*It is possible to shower/bathe with the product on, covering the foot in such a way that neither the discharge nor the wound gets wet (e.g., plastic bag).*

*The duration of treatment will vary from patient to patient.*

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### **PAO (PERIOD AFTER OPENING)**

*Do not apply. There is no caducity or changes on the product composition due to open the packaging.*

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### **EMERGENCY AND FIRST AID PROCEDURES**

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##### ***Eye contact***

*Is a solid product. In case the thin layer of talcum powder that has on its surface case irritation on the eye flush with water holding eyelids open.*

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##### ***Skin contact***

*The product is used on direct contact with the skin. We recommend not use the product more than 14-16 hours a day to allow transpiration of the skin.*

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**Inhalation**

*Is a solid product. No risk of inhalation.*

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**Ingestion**

*Medical oils used are not toxically, but in case of ingestion we recommend to get medical advice.*

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**Anti-inflammable measures**

*Not flammable but will burn. Combustion products may include carbon monoxide and carbon dioxide.*

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**Extinguishing media**

*Foam, water spray or fog. Dry chemical powder, carbon dioxide, sand or earth may be used for small fires only. Do not use direct water jet on burning material.*

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**HANDLING**

*The products are in direct contact with the skin on its normal use, nonspecific cares should be taken neither is needed to wear gloves for its manipulation.*

*Risks of electrostatic accumulation: not applicable.*

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**STORAGE**

*Temperature ambient. Do not stock it on higher temperatures than 40°C, its final shape can vary.*

*Keep container dry.*

*Preserve in a cool, well-ventilated place.*

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No transportar a temperaturas superiores a 60°C.

*Transportation temperature: Ambient.*

*Do not transport it on higher temperatures than 60°C.*

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*Transportation pressure: Atmospheric*

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**CONTROL OF INDIVIDUAL PROTECTION**

*Not specific cares should be taken on normal use.*

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### **MEASURES TO FOLLOW IN CASE OF A FORTUITE ACCIDENT**

*Fresco International 2005, S.A. must be notified of any serious incident that has occurred in relation to the products declared in this document, using the information provided in the cover of this document.*

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### **IMPORTANT INFORMATION**

*The information provided should be regarded as a description of the relevant requirements of the safety of the product, it does not constitute a guarantee and does not replace the work or opinion of the Medical Professional.*

*Fresco International 2005, S.A., does not accept responsibility for misuse of the manufactured products, or if used for a different activity, the conditions of its use are not under Fresco Internationals 2005, S.A. control.*

## 21. CONTROL DE CAMBIO

### **DOCUMENT VERSION CONTROL**

| Versión<br>Version | Descripción del cambio<br>Description of changes made to the doc. | Autor<br>Author | Fecha efectiva<br>Effective from |
|--------------------|-------------------------------------------------------------------|-----------------|----------------------------------|
| 00                 | Elaboración del documento<br>Document Development                 | QA/RA Director  | 25/09/2011                       |
| 01                 | Adaptación a MDR<br>MDR adaptation                                | QA/RA Director  | 24/09/2021                       |

**FRESCO INTERNATIONAL 2005, S.A.**  
N.º F.º A63702633  
Pol. Ind. Can Vellat  
C/ Torre de Gellars, 5, naves 3 y 4  
08150 Parets del Vallès (Barcelona)  
Tel. 93 573 94 00

**Yanina Trombino**  
Responsable de la Regulación  
*Regulatory Responsible*

**FRESCO INTERNATIONAL 2005, S.A.**  
N.º F.º A63702633  
Pol. Ind. Can Vellat  
C/ Torre de Gellars, 5, naves 3 y 4  
08150 Parets del Vallès (Barcelona)  
Tel. 93 573 94 00

**Aitor Fresco**  
Director general  
*General Director*